ALUMINIUM WALKING STICK

GB IE
ALUMINIUM WALKING STICK
 Instructions for use

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
IAN 96822

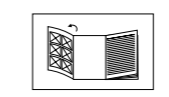IAN 96822

GB IE
 Before reading, fold out the illustration page and get to know all of the functions of your unit.

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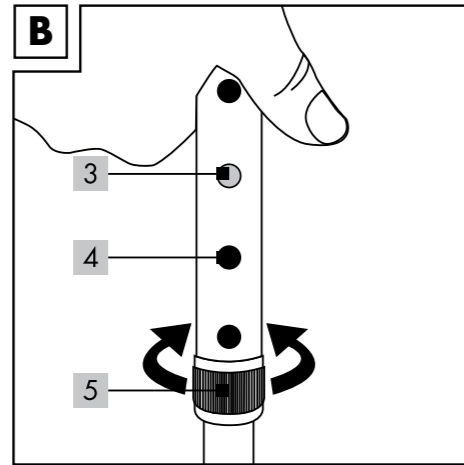
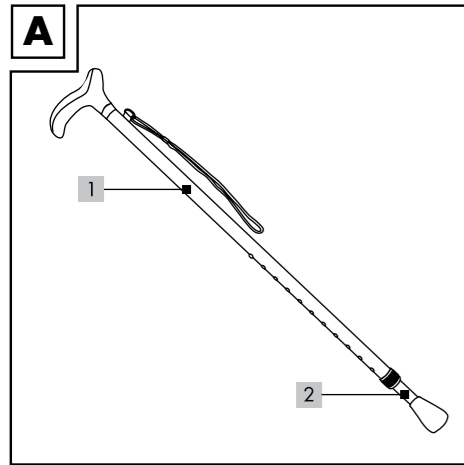
Congratulations!

With your purchase you have decided on a high-quality product. Get to know the product before you start to use it. Carefully read the following operating instructions. Use the product only as described and only for the given areas of application. Keep these instructions safe. When passing the product on to a third party, always make sure that the documentation is included.

This product conforms to guideline 93/42/EWG on medical devices.

Contents & product overview

- (1) Walking stick
- (2) Telescopic tube
- (3) Height adjustment button
- (4) Drill holes in the walking stick
- (5) Adjustment

Intended Use

The walking stick is intended as a walking aid. It is designed for private indoor and outdoor use.

Technical Specifications

Length: telescopic tube adjustable from 750mm - 1000mm

Weight: approx. 357g

Max. user weight: 100kg

Symbols used

Observe the instructions

Date of manufacture

Batch designation

Manufacturer

For use outdoors and in enclosed spaces

Safety Instructions

Risk of injury!

- Keep children away from the packaging material. Risk of suffocation!
- Keep children away from the walking stick. It is not a toy!
- Inspect the item regularly for signs of damage. Do not use if it has visible signs of damage. Only have it repaired by a qualified specialist.
- Do not make any modifications to the item and do not repair it yourself! Incorrect repairs can cause unforeseeable damage! In this case, contact our Product Service.
- Do not use the walking stick as a truncheon, etc. Risk of injury!
- Do not place your finger into the height adjustment drill holes and ensure your skin does not get pinched. Risk of injury!
- Always walk with the walking stick on a firm surface, e.g. plastered or tarmac surfaces. On soft ground the walking stick could sink and you could lose your footing!
- Use this article only when surface temperatures are suitable for its use. Hot water, direct sunlight, sauna conditions, extreme cold, etc. can cause the surface of the article to assume temperatures which could endanger health.

Avoid damage!

- Do not use any harsh or abrasive cleaning agents.

Adjusting the walking stick to the required height

CAUTION!

Do not use the item if there are visible signs of damage! Ensure that you do not pinch your skin in the drill holes.

Do not use the walking stick if the height adjustment button has not engaged in the drill hole!

1. Remove the packaging.
2. Push the button (3) on the walking stick and extend the telescopic tube (2) to the required height.
3. The telescopic tube must noticeably engage in the required position. It is only correctly engaged if the button is visible at the drill hole (4) on the walking stick again. If it is not possible to engage the walking stick immediately, pull the telescopic tube out slightly until it engages.

Adjusting the Walking Stick

Turn the adjuster (5) clockwise to fix the height. Turn it ant-clockwise to loosen it.

Cleaning

CAUTION!

Do not use any harsh or abrasive cleaning agents or sponges.

These can damage the surface.

1. If necessary, wipe over the walking stick with a slightly damp cloth and a small amount of mild detergent if necessary.
2. Dry it completely before you use it again.

CAUTION!

Do not make any modifications to the product and do not repair the item yourself! Only obtain spare parts from the manufacturer or the wholesaler. Incorrect repairs can cause unforeseeable damage!

Disposal

Dispose of the packaging in an environmentally friendly manner. Dispose of it where valuable waste is collected. Dispose of the product through an approved disposal company or the municipal disposal facilities. Observe the current regulations. If in doubt, consult your local authority about an environmentally friendly disposal.

3 years warranty

The product was produced with great care and under constant supervision. You receive a three-year warranty for this product from the date of purchase. Please retain your receipt. The warranty applies only to material and workmanship and does not apply to misuse or improper handling. Your statutory rights, especially the warranty rights, are not affected by this warranty.

With regard to complaints, please contact the following service hotline or contact us by e-mail. Our service employees will advise as to the subsequent procedure as quickly as possible. We will be personally available to discuss the situation with you.

Any repairs under the warranty, statutory guarantees or through goodwill do not extend the warranty period. This also applies to replaced and repaired parts.

Repairs after the warranty are subject to a charge.

IAN: 96822

 Service Great Britain

Tel.: 0871 5000 720
(£ 0.10/Min.)

E-Mail: deltasport@lidl.co.uk

 Service Ireland

Tel.: 1890 930 034
(0,08 EUR/Min., (peak))
(0,06 EUR/Min., (off peak))

E-Mail: deltasport@lidl.ie

You can also find spare parts for your product at: www.delta-sport.com, category Service - Lidl Spare Parts Service

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