BACK BRACE PRO COMFORT

GB IE

BACK BRACE

Instructions for use

SE

RYGGBANDAGE

Bruksanvisning

FI

SELKÄTUKISIDE

Käyttöohje

DK

RYGBANDAGE

Bruksanvisning

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

For personal use only. Consult a doctor if:

- you are unsure how to use the product
- you have acute injuries
- unforeseen problems arise

Consult a doctor before wearing the brace for hours at a time and do not wear in combination with other medical products, e.g. creams or salves.

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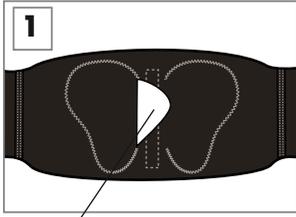
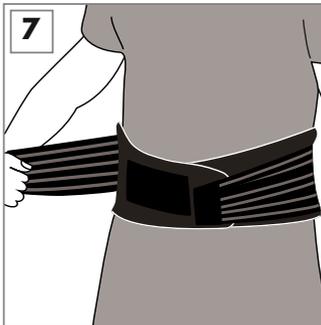
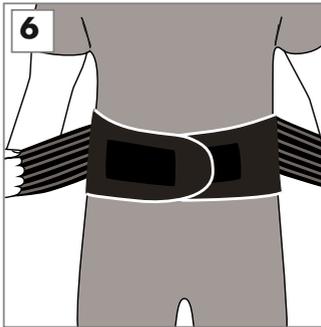
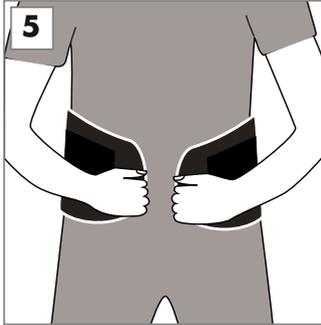
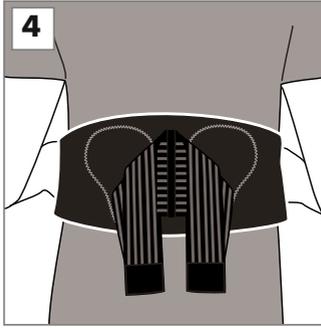
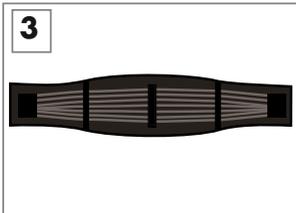
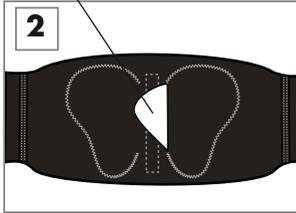
GB IE

Applying the brace

If you can feel that the lumbar pads are resting above your buttocks, then the brace is correctly positioned. (The pads can also be optionally removed.)

For additional stabilisation of the lumbar spine use the compression strap, which can be applied optionally using the Velcro fastener. The more the strap is stretched as it is put on, the greater the stabilisation.

Images 1 and 2: Lumbar pads (can be optionally removed)

Image 3: Velcro fastener and compression strap

Image 4: The side Velcro fasteners of the compression strap must be unfastened before putting on the brace.

Image 5: After the brace has been fastened it should feel comfortable and slightly taut.

Image 6: It is best to stretch the compression strap using both hands.

Image 7: Fasten the side Velcro fasteners of the compression strap one at a time.

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Horizonte Textil GmbH, Am Sandtorkai 23, D-20457 Hamburg