KNEE BRACE PRO COMFORT

GB IE

KNEE BRACE

Instructions for use

FI

POLVITUKISIDE

Käyttöohje

SE

KNÄLEDSBANDAGE

Bruksanvisning

DK

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Brugsanvisning

IAN 93468

GB IE FI SE DK

Congratulations on the purchase of your new product. You have chosen a high-quality article. The instructions for use are part of this product. They contain important information on safety and use. Please familiarise yourself with the instructions and safety information before using this product. Use the product only in perfect condition, as described and for the specified area of use. Please keep all packaging including this leaflet for as long as you use the brace and be sure to include them when handing on the product to another person.

For personal use only. Consult a doctor if:

- you are unsure how to use the product
- you have acute injuries
- unforeseen problems arise

Consult a doctor before wearing the brace for hours at a time and do not wear in combination with other medical products, e.g. creams or salves. Ensure good circulation when wearing the brace!

	Consult instructions for use
	Manufacturing date
	Batch designation
	Manufacturer

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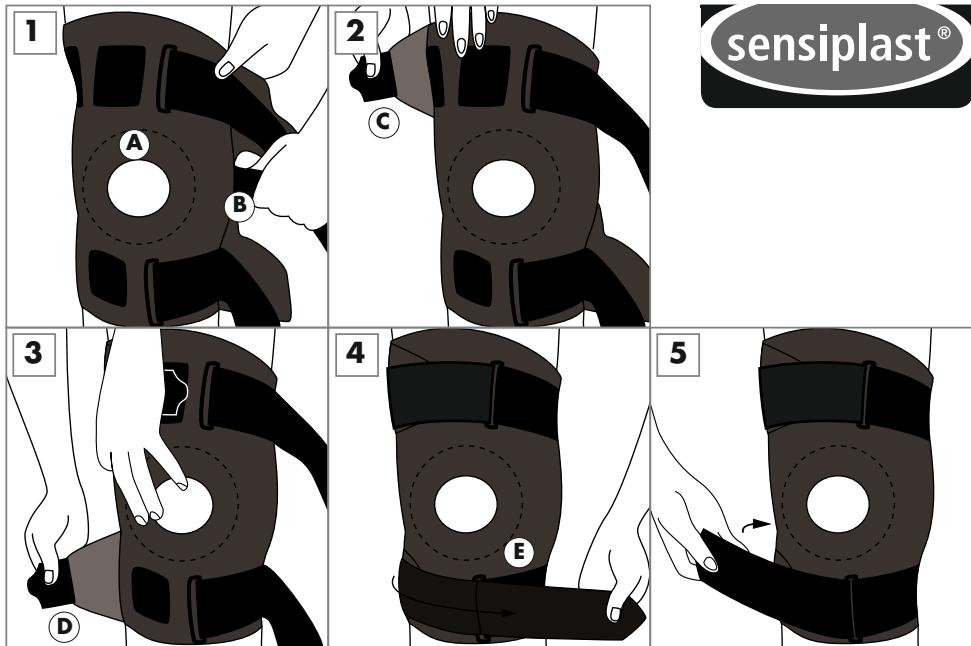
(GB) (IE)

Applying the brace

It is best to put on the brace when sitting. To achieve optimal support, the brace should be applied as tautly as possible, but without being restrictive.

Image 1: Open the brace and place it on the knee so that the patella ring is positioned exactly over the kneecap. Slightly bend the knee and close the middle Velcro fastener.

Images 2 and 3: Close the top and bottom Velcro fasteners.

Images 4 and 5: Pull the bottom stabilisation strap around the back of the knee and brace and pull it through the buckle opening before securing it with the Velcro fastener. Repeat in exactly the same way with the upper stabilisation strap.

- A: Patella ring
- B: Middle Velcro fastener
- C: Top Velcro fastener
- D: Bottom Velcro fastener
- E: Buckle for the stabilisation strap

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Horizonte Textil GmbH, Am Sandtorkai 23, D-20457 Hamburg