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GB COLD COMPRESS -

IE INFORMATION AND DIRECTION FOR USE

sensiplast® Cold Gel Compress

Congratulations on purchasing this sensiplast® Cold Gel Compress. You have acquired a high-quality medical product for cold therapy.

 Please read this information and direction for use carefully and retain it.

 Always keep warnings in mind.

Cold therapy:

e.g. headache, migraine, itching, nosebleed, bruises, fever, insect bites, toothache, contusions, sprains and burns.

 Do not use in cases of peripheral circulatory disorders, Raynaud’s syndrome, restricted temperature sensation and when taking painkillers. Consult a physician before use in cases of circulatory problems and other pre-existing illnesses.
For external application only. Not suitable for damaged skin or open wounds.

 Store in a refrigerator or freezer for about 1 hour

 Place the cold compress in the accompanying cover

 Place the cold compress on the area to be treated

 Risk of local frostbite if used without fabric cover or cloth.

 Inspect before use. Do not continue to use damaged compress. Dispose of it with domestic waste. Keep away from naked flames. In case of contact, rinse off the affected skin areas under running water. Consult a physician if swallowed.

 Keep out of the reach of children. Do not use for children under the age of 3. Only use for older children under supervision.

Cleaning the cold compress:

Use a mild soap to clean under running, lukewarm water after use. Do not use any cleaning agents containing solvents such as disinfectant. Dry with a soft cloth or leave to air dry.

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